



TRIODERM®

## Trioderm® CARE Cryospray for wart removal, 50 ml

---

### Trioderm® CARE Cryospray for wart removal

Medical device.

#### General description

Trioderm® CARE Cryospray for wart removal is designed to remove small skin warts (up to approx. 5 mm in diameter) by quickly freezing the treated area using liquefied gases with a low boiling point by attaching a frozen foam sponge.

The spray is designed for home use, the application is simple, fast and safe. Also suitable for children from 4 years of age.

#### Intended purpose

Removal of smaller skin warts (up to 5 mm in diameter) by quickly freezing the treated area by applying a frozen foam sponge.

#### Indications

Occurrence of skin warts up to about 5 mm in diameter.

#### Contraindications

Do not spray the product directly on the wart or on the skin to prevent frostbite. Do not apply on mucous membranes. Do not use around eyes, on ears, sensitive areas of the body or open wounds! Do not use the spray to treat the same wart more than 3 times! Do not use the spray on skin lesions that you are not sure are warts. Do not use if you are allergic to any of the ingredients of the product. Do not use for children under 4 years of age. Consult a doctor for use in diabetics, people with circulatory disorders, pregnant women and women who are breastfeeding. In case of persistent pain after treatment of warts with the spray, stop using the spray and consult your doctor.

#### Intended users

Cryospray for wart removal is intended for home use. Not suitable for children under 4 years of age. Consult a doctor before using the spray on diabetics, people with circulatory disorders, women during pregnancy and breastfeeding.

#### Intended site of application

The surface of the skin wart, by applying a frozen foam sponge.

#### Duration

Transient effect for the duration of applying a frozen foam sponge to the wart.

#### Directions for use

Freezing can be painful in the short term. To increase the effectiveness before treatment, soften the wart by soaking it in the water for 5 minutes or by applying a wet cosmetic bud for the indicated time.

1. Place the foam sponge (roller) with the darker end onto the attached plastic tube and insert the tube into the mouth of the applicator.
2. Press the applicator for 2 seconds.
3. Wait 20 seconds until the foam sponge is saturated with the active substances and freezes.
4. Grip the extension, pull it out of the mouth of the applicator and place the frozen foam sponge as fast as possible on the centre of the wart.
5. Gently push the foam sponge down on the wart and allow it to act for 10-40 seconds depending on the type of wart (see attached table and figure).
  - a. 10 s for warts on the hands and other parts of the body except the feet under 2.5 mm in diameter
  - b. 15 s for warts on the hands and other parts of the body except the feet under 5 mm in diameter
  - c. 40 s for warts on the feet
6. Remove the foam sponge from the wart.
7. For effective treatment, the entire wart should be frozen, including a 1-2 mm circle of skin around the wart (creating a white circle) to ensure that the entire lesion is removed.
8. Dispose of used foam sponges, discard.
9. If necessary, repeat the treatment after 2 weeks (after point 1). Treat one wart a maximum of 3 times in total, 2 weeks apart from each application.

#### Specific safety requirements for an intended purpose

After use, a blister may form at the treatment site or a temporary change in skin colouring, namely hypopigmentation (lightening) or hyperpigmentation (darkening) of the skin at the treatment site. Long-term freezing may result in temporary loss of sensitivity of the treated site. Treating a wart on the forehead or on the temple can cause headache. Longer treatment of warts on a hair part of the head can result in permanent loss of hair. Longer freezing of the warts around the nail may result in a change in the shape or structure of the nail. May cause an allergic skin reaction in hypersensitive individuals. In case there is severe reddening of the treated site, frostbite or a big blister, and in case of doubt always seek out a doctor. Protect your eyes from the spray. Do not inhale the spray. Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, an open flame or other sources of ignition. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children. Do not eat, drink or smoke while using the spray. Any serious adverse event associated with this product should be reported to the manufacturer and the State Institute for Drug Control. Dispose of in organized waste collection.



TRIODERM®

## Trioderm® CARE Cryospray for wart removal, 50 ml

---

### Shelf life

24 months. Use by the date indicated on the bottom of the container.

### Ingredients

Butane, propane, dimethyl ether, isobutane

**Text revision date: March 2022**



AVEFLOR, a.s., Budčevs 26, 507 32 Kopidlno,  
Czech Republic, [www.akutol.cz](http://www.akutol.cz)



Hazard

