



TRIODERM®

## Trioderm® CARE Cooling Spray, 200 ml

---

### Trioderm® CARE Cooling Spray

Medical device for quick pain relief. Also suitable for children.

#### General description

Trioderm® CARE Cooling Spray is a cooling spray for quick pain relief associated with muscle contusion, joint sprain, muscle spasms and other injuries due to an accident, fall or sprain and for pain relief during minor procedures such as venepuncture when inserting a cannula or catheter into a vein. Contains low-boiling substances acting on the affected site like a local anaesthetic. It creates an immediate cooling effect at the site of treatment, relieves the feeling of pain and relaxes the affected site for 3 minutes.

Suitable for the whole population, children from 3 years of age, for diabetics.

#### Intended purpose

Transient pain relief for muscle bruising, joint sprains, muscle spasms and other injuries due to impact, falls or increased sports activity and pain relief for minor procedures such as venepuncture before insertion of a cannula or catheter into a vein.

#### Indications

Pain associated with muscle contusion (Musculus contusus), joint sprain (luneturam distorsio), with muscle spasms (Musculus distentio) and other minor sport injuries due to an accident, fall or increased activity (Athletae effectus auctus) and pain accompanying minor procedures such as venepuncture before the insertion of a cannula or catheter into a vein.

#### Contraindications

Do not spray for a longer time and from a shorter distance to avoid frostbite. Do not apply to the face, neck, sensitive areas or open wounds. Not to be used in case of allergic reaction to any of the contained ingredients. Do not use in children under 3 years of age. Consult the use of the spray in pregnant and breast-feeding women with a doctor.

#### Intended users

For the entire population, for children from 3 years of age and for diabetics.

#### Intended site of application

Intact skin on the affected areas of the body.

#### Duration

Transient effect within 3 minutes.

#### Directions for use

The spray is applied by pressing the sprayer on the skin over the affected site at a distance of 15 cm from the body surface, it is applied 2-3 times by a repeat 5 second spray.

#### Specific safety requirements for an intended purpose

May cause an allergic skin reaction in hypersensitive individuals. In case there is severe reddening of the treated site, frostbite or blisters after excessive use of the spray and doubt, always seek out a doctor. Protect your eyes from the spray. Do not inhale the spray. Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, an open flame or other sources of ignition. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children. Any serious adverse event associated with this product should be reported to the manufacturer and the State Institute for Drug Control. Dispose of in organized waste collection.

#### Shelf life

30 months. Use by the date indicated on the bottom of the container.

#### Ingredients

Butane, propane, isobutane

Text revision date: April 2021




AVEFLOR, a.s., Budčevs 26, 507 32 Kopidno,  
Czech Republic, [www.akutol.cz](http://www.akutol.cz)



Hazard

3

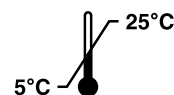

Czech  
product