



TRIODERM®

Trioderm® CARE Ear Spray, 50 ml

Trioderm® CARE Ear Spray

Water-based medical device containing herbal substances for ear hygiene – for all age categories, for children from 3 years of age and for diabetics.

General description

Trioderm® CARE Ear Spray helps restore earwax and preserve its protective and hydrating function. Helps clean the ear, improves ear health and preventively reduces inflammation. The gentle spray stream has a mechanical function, the compressed air evenly disperses microscopic aerosol droplets in the external auditory canal. The spray solution flushes away dust, dirt and microorganisms present. It softens earwax, adds moisture and elasticity to the skin of the external auditory canal and thus helps prevent inflammation. The spray helps to restore earwax and maintain its protective, hydrating and cleansing function, and prevents formation of ear plugs when used regularly. In case of ear inflammation, it is possible to use Trioderm® CARE Ear Spray as a supportive agent for medicated treatment. It forms a moist microenvironment in the external auditory canal, which is suitable for accelerating the regeneration of the affected area.

Indications

Excessive amount of earwax in the external auditory canal, ear hygiene, prevention of ear plugs, preventive reduction of inflammation and ear pain, supportive agent in the medicated treatment of ear inflammation.

Contraindications

Do not use in case of an allergic reaction to any of the ingredients of the product. If you have had paracentesis (puncture of the ear drum), use the spray only under medical supervision or only subject to medical recommendation! In the following cases, consult your ENT specialist: total blockage of the auditory canal by earwax, infection and pain in the ear or auditory canal, balance disorders, immunosuppression, surgery of the or auditory canal, ear drum rupture, foreign object in the auditory canal, pregnancy and lactation. Protect your eyes from the spray. Do not inhale the spray.

Intended purpose

Softening of earwax, adding moisture and elasticity to the skin of the external auditory canal, creating a moist microenvironment that helps to speed up the regeneration of the treated area.

Intended users

For the entire population, for children from 3 years of age and for diabetics.

Intended application

External auditory canal

Directions for use

1. Before application, gently bend your head: to the right to apply to the left ear and vice versa. It is not necessary to keep the spray in an upright position during application.
2. Carefully insert the applicator into the outer part of the ear. The tip of the dispenser is specifically shaped to prevent the introduction of the applicator deep into the ear and damage to the ear drum.
3. Press the applicator for 1 second but do not spray directly against the ear drum! Keep your head tilted for at least 30 seconds so the fluid can spontaneously flow into the inner part of the auditory canal.
4. After duration of action, tilt your head to the other side and wait until the solution flows out.
5. Carefully dry the outer parts of the ear with a clean towel or tissue paper.
6. Repeat the whole process for the other ear.
7. Wash the applicator with hot water and soap after each use. Due to hygienic reasons, it is recommended to use one spray per person.
8. Dosage: 1–3 times a week, every day if needed.

Specific safety requirements for an intended purpose

Any serious adverse event associated with this product should be reported to the manufacturer and the State Institute for Drug Control. Dispose of in organized waste collection. May cause an allergic skin reaction in hypersensitive individuals.

Specific safety requirements

Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, an open flame or other sources of ignition. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C. Keep out of reach of children.

Shelf life

Use by the date indicated on the bottom of the container.

Ingredients

Aqua, Propylene Glycol, Polysorbate 80, Aloe Ferox Leaf Extract, Malva Sylvestris Extract, Phenoxyethanol, Ethylhexylglycerin, Tocopheryl Acetate, Scent.

Text revision date: January 2020



AVEFLOR, a.s., Budčevs 26, 507 32 Kopidno,
Czech Republic, www.akutol.cz




TRIODERM®

Trioderm® CARE Ear Spray, 50 ml



3

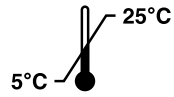

Czech
product

Hazard