



TRIODERM®

Trioderm® CARE Foam for very dry skin, 150 ml

Trioderm® CARE Foam for very dry skin

Medical device - foam for treatment of very dry, sensitive and scaly skin or as complementary care for skin affected by psoriasis or atopic dermatitis. Suitable for treatment of very dry parts of the body such as elbows, knees, soles or heels. It makes the skin softer, included substances heal dry parts on the body and supply necessary moisture and softness. The complex effect of the ingredients it contains brings relief, soothes and alleviates the affliction.

General description

Trioderm® CARE Foam for very dry skin creates a protective film on the skin surface after spreading and hydrates the treated area. The water content is an essential factor for maintaining the smoothness and elasticity of the skin. Trioderm® CARE Foam for very dry skin restores the skin protective barrier function by saturating it with water and fatty acids. The ingredients of the foam hydrate the treated area from the inside and create a mechanical barrier on the surface, which prevents moisture loss in skin. After the treatment, the skin becomes softer, the contained ingredients soothe the treated area and support the restoration of the skin protective barrier function. Naturally slightly acidic pH level prevents propagation of microorganisms and inflammation. The recovery process of the affected dry area is supported by the presence of vitamin E.

- It contains 8% of vegetable oils (evening-primrose oil, apricot kernel oil, macadamia oil) and 6% of emollients which overlap, hydrate and soften the affected area.
- It contains 15% of Natural Hydrophilic Factors, natural hydrophilic substances which bind water and hydrate the dry skin from inside.
- It contains no colourants or paraffin oil; it is delicately scented and free of allergens.
- The acidic pH encourages the natural protective function of the skin.

Intended purpose

Foam for the treatment of very dry, sensitive and scaly skin or as complementary care for skin affected by psoriasis and atopic dermatitis.

Indications

Foam for the treatment of very dry, sensitive, scaly skin or as additional care for skin affected by psoriasis (Psoriasis vulgaris) and atopic dermatitis (Dermatitis atopica). Suitable for treatment of very dry parts of the body such as elbows, knees, soles or heels.

Contraindications

Not to be used in case of an allergic reaction to any of the ingredients of the product. If in doubt, consult your doctor.

Intended users

Trioderm® CARE Foam for very dry skin can be used for the whole population, for children from 3 years old and for diabetics. Use in children under 3 years of age, pregnant and breastfeeding women should be consulted with a physician.

Intended site of application

Very dry areas of the body, such as elbows, knees, feet, heels, and areas affected by psoriasis or atopic dermatitis.

Directions for use

1. Apply the foam by pressing the sprayer to a hand or affected area, gently spread by hand.
2. Apply the foam preferably immediately after showering or washing while the skin is slightly damp.
3. Let the foam get absorbed by your skin, do not wash it off. Repeat the treatment as needed several times a day, application 3 times a day is usually sufficient. For reasons of hygiene, rinse the sprayer with running hot water after use and cover with a protective cap.

Any serious adverse event associated with this product should be reported to the manufacturer and the State Institute for Drug Control.

Duration

Use until the condition of the skin improves, or as recommended by the attending physician.

Specific safety requirements for an intended purpose

Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, an open flame or other sources of ignition. No smoking. Do not pierce or burn, even after use. Do not spray into open flames or other sources of ignition. Protect from sunlight. Do not expose to temperatures exceeding 50 °C. Keep out of reach of children. Protect your eyes from the spray. Dispose of the container by handing over to a collection point as hazardous waste.

Shelf life

Use by the date indicated on the bottom of the container.

Ingredients

Aqua, oenothera biennis oil, hydroxyethyl urea, butane, glycerin, caprylic/capric triglyceride, prunus armeniaca kernel oil, cetareth-25, cetearyl alcohol, sodium lactate, propane, macadamia ternifolia seed oil, c12-15 alkyl benzoate, lactic acid, isobutane, tocopheryl acetate, phenoxyethanol xanthan gum, scent, ethylhexylglycerin

Text revision date: September 2020



TRIODERM®

Trioderm® CARE Foam for very dry skin, 150 ml



AVEFLOR, a.s., Budčeves 26, 507 32 Kopidlno,
Czech Republic, www.akuto1.cz



3

